



Sample Collection Instructions

1 – COLLECT MOTHER’S BLOOD

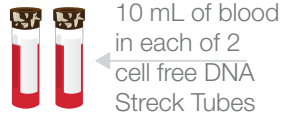


- Fill both tubes completely. If insufficient volume is obtained, please draw an additional tube.
- Allow 60-90 seconds for each tube to fill.
- Please use 21 gauge straight needle, **NOT** butterfly
- Vein collapse may require second venipuncture with a fresh tube.

2 – GENTLY MIX SAMPLE

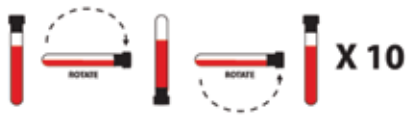


- Gently invert each tube at least 10 times immediately after draw in order to thoroughly mix blood with reagents.
- Do not shake vigorously.
- Do not seal tubes with paraffin film

3 – PRE-PACK SAMPLE

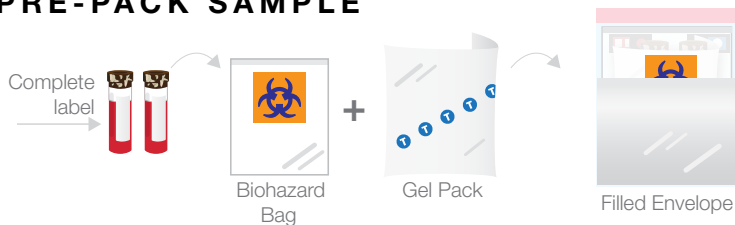


- A Complete information on tube labels.
- B Place inside provided biohazard bag.
- C Place in between gel pack to cushion.
- D Place gel pack in metallic envelope.

4 – SHIP SAMPLE

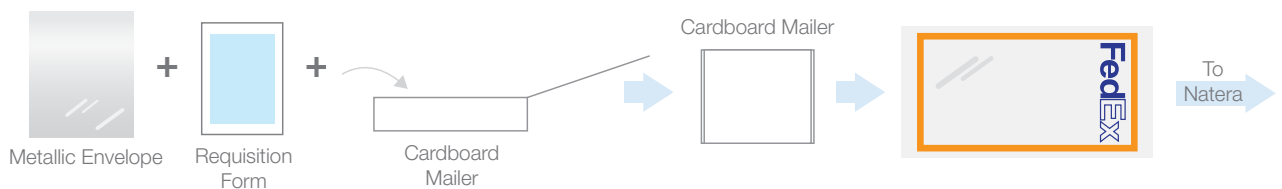

- A Place metallic envelope with the blood samples and requisition form into original Panorama Kit (cardboard mailer).
- B Place cardboard mailer inside FedEx Clinical Pak.
- C Call FedEx at 1-800-GO-FEDEX to arrange pickup; prepaid FedEx label is affixed to Clinical Pak.
- D Do **NOT** expose blood to temperatures outside the range of 43-98 °F or 6-37 °C.
- E Blood should never be refrigerated or frozen
- F Do **NOT** store kits in an area where the temperature range is outside 65-86 °F or 18-30 °C.

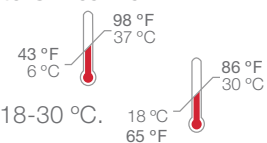

This test was developed by Natera, Inc. a laboratory certified under the Clinical Laboratory Improvement Amendments (CLIA). This test has not been cleared or approved by the U.S. Food and Drug Administration (FDA). Although FDA does not currently clear or approve laboratory-developed tests in the U.S., certification of the laboratory is required under CLIA to ensure the quality and validity of the tests.
© Natera 2015 All Rights Reserved