



Prospera™

Transplant assessment

Sample Collection Instructions

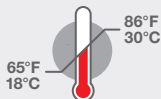

DO NOT store kits in an area where the temperature range is outside 65°F–86°F (18°C–30°C).

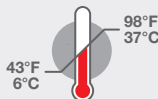

DO NOT expose blood to temperatures outside the range of 43°F–98°F (6°C–37°C).


1. Collect the patient's blood



10 mL of blood
in each of two Streck
cell-free DNA tubes

- Fill both tubes completely. If insufficient volume is obtained, please draw an additional tube.
- Allow 60–90 seconds for each tube to fill.
- Use a 21 gauge straight needle.
- Vein collapse may require a second venipuncture with a fresh tube.
- **DO NOT USE A BUTTERFLY NEEDLE.**

2. Gently mix the sample

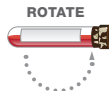


x 10



- Gently invert each tube at least 10 times immediately after draw in order to thoroughly mix blood with reagents.
- **DO NOT** shake vigorously.
- **DO NOT** seal tubes with paraffin film.

Continued on back

3. Pre-pack the sample

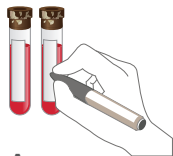

A

Complete information on tube labels.




B

Place tubes inside provided biohazard bag.



C


Place the bag inside gel pack to cushion.

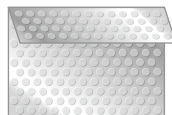

D

Place the gel pack in the metallic envelope.

4. Ship the sample

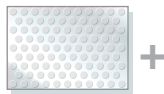

E

Place the metallic envelope with the blood samples and requisition form into the original kit (cardboard mailer).



F

Place the cardboard mailer inside the FedEx Clinical Pak.



G

Call Natera's Prospera Customer Care team at 650.273.4468 for questions on shipping matters. Options include FedEx pick up/drop off* (9:30AM next day) or Rush Courier (same day deliver)**



*9:30 AM next day is dependent on package making previous days' pick up window

**1-2 hr pick up window depends on pickup location

This test was developed by Natera, Inc., a laboratory certified under the Clinical Laboratory Improvement Amendments (CLIA). This test has not been cleared or approved by the US Food and Drug Administration (FDA). Although FDA does not currently clear or approve laboratory-developed tests in the US, certification of the laboratory is required under CLIA to ensure the quality and validity of the tests. MLB-10096 Rev01 Prospera Domestic Sample Collection Instructions-NAT-801960 © 2019 Natera, Inc.

T: 650.273.4468

W: www.natera.com/prospera